



BV symptoms RELIEF & TREATMENT

**1 APPLICATION = UP TO 3 DAYS OF RELIEF
LONG LASTING
CLINICALLY TESTED
PERFUME FREE**

Replens™ BV symptoms RELIEF & TREATMENT gel helps maintain a healthy vaginal pH. A single application of Replens™ BV symptoms RELIEF & TREATMENT gel can maintain the vaginal pH in the normal range for 72 hours or more and gives long-lasting freshness.

- Eliminates feminine odour
- Relieves itching & irritation
- Treats and prevents bacterial vaginosis
- Maintains healthy pH to help reduce the risk of vaginal infections

Please read this leaflet carefully before using Replens™ BV symptoms RELIEF & TREATMENT gel. Keep the leaflet, you may need to read it again. If you have any questions or are not sure about anything, ask your doctor or pharmacist. In this leaflet:

1. What Replens™ BV symptoms RELIEF & TREATMENT gel is and what it is used for
2. What can cause Bacterial Vaginosis (BV)
3. Warnings. Read before you use Replens™ BV symptoms RELIEF & TREATMENT gel
4. Directions for use of Replens™ BV symptoms RELIEF & TREATMENT gel
5. How Replens™ BV symptoms RELIEF & TREATMENT gel works
6. Frequent questions about Replens™ BV symptoms RELIEF & TREATMENT gel
7. Discover other Replens™ products

1. What Replens™ BV symptoms RELIEF & TREATMENT gel is and what it is used for

Replens™ BV symptoms RELIEF & TREATMENT gel is a soft clear intravaginal gel. It is a medical device used for the relief and treatment of Bacterial Vaginosis (BV) symptoms such as unusual vaginal odour (usually fishy smell), excessive discharge (usually watery or greyish), irritation, itch and discomfort. One single application provides up to 3 days of long lasting relief. Replens™ BV symptoms RELIEF & TREATMENT gel is clinically tested and contains no perfume, no dye, no drugs and no hormones.

2. What can cause Bacterial Vaginosis (BV)

BV may occur when vaginal pH is unbalanced and bacteria can flourish. This imbalance can be triggered by many factors such as:
- unprotected sexual intercourse or a change of partner
- hormonal changes (menstrual periods, pregnancy, menopause, ...)
- douching or using perfumed products in or around the vagina
- stress


3. Warnings. Read before you use Replens™ BV symptoms RELIEF & TREATMENT gel

KEEP OUT OF REACH OF CHILDREN. Not for use by children. Keep out of eyes and ears. If pregnant or breastfeeding, consult healthcare provider before use. This product is not a contraceptive and does not contain spermicide. If vaginal irritation occurs, discontinue use. If symptoms persist, consult a doctor. Replens™ BV symptoms RELIEF & TREATMENT gel is compatible with natural rubber latex condoms and polyisoprene condoms but is NOT compatible with polyurethane condoms. Each applicator is individually wrapped. DO NOT USE this product if an applicator is unwrapped or the wrapper is torn or missing. DO NOT re-use. Apply immediately after opening. For intra-vaginal use only.


4. Directions for use of Replens™ BV symptoms RELIEF & TREATMENT gel

Use at first signs or use as prevention. Apply every 3 days as needed.


1. Remove the applicator from the sealed wrapper. DO NOT open the applicator at this time.
2. Grip the applicator firmly by the thick end. Shake down 4 times to ensure that the contents are at the thin end (see Figure 1).

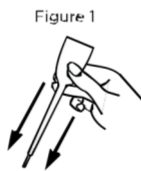


3. Bend the thin pointed end of the applicator back and forth repeatedly. Snap off the thin pointed end of the applicator and discard the break-away tip (See Figure 2).
*DO NOT use scissors or tools to cut the applicator tip.

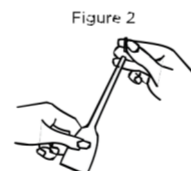

4. While sitting or lying on your back with knees bent, gently insert the open end of applicator well into the vagina (see Figure 3).

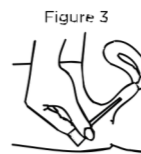

5. Squeeze the thick end of the applicator firmly 2-3 times to deposit the gel (see Figure 4).
6. Remove the applicator and discard in waste container.

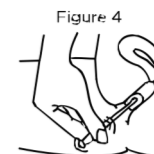

Note: The applicator contains 7.8g of gel and is designed to deposit the recommended amount of gel (2g). When used as directed, a small amount of gel will remain in the applicator after use. Discard this excess product along with the applicator and do not attempt to reuse the applicator.

Using Replens™ BV symptoms RELIEF & TREATMENT gel maintains a healthy vaginal pH to help reduce the risk of vaginal infections.

5. How Replens™ BV symptoms RELIEF & TREATMENT gel works

A healthy vaginal pH ranges between 3.5 and 4.5. When pH is 'healthy' or 'balanced', the vaginal environment is characterised by a low number of bad bacteria and a high number of good bacteria. When pH gets too high (for instance, after unprotected intercourse as semen level is about 7), the vaginal pH can be imbalanced: bad bacteria flourish and good bacteria die. That's when BV symptoms can occur.

Replens™ BV symptoms RELIEF & TREATMENT gel is clinically tested to balance vaginal pH. It has a unique bio-adhesive complex (the Polycarbophil and Carbomer System). The LONG LASTING formula adheres to the vaginal wall (epithelium) for up to three days and during this time it continuously releases pH balancing ions that rapidly and effectively reduce the pH to a healthy level. Replens™ BV symptoms RELIEF & TREATMENT gel effectively relieves and treats BV symptoms such as odour, discharge, irritation, itch and discomfort.

6. Frequent questions about Replens™ BV symptoms RELIEF & TREATMENT gel

Is Replens™ BV symptoms RELIEF & TREATMENT gel safe?

Yes, Replens™ BV symptoms RELIEF & TREATMENT gel has been tested for its safety. Also, intercourse is safe while using the product. Replens™ BV symptoms RELIEF & TREATMENT gel is compatible with natural rubber latex condoms and polyisoprene condoms but is NOT compatible with polyurethane condoms.

Is Replens™ BV symptoms RELIEF & TREATMENT gel messy?

No. The pre-filled disposable applicator keeps the mess to a minimum, and the amount of Replens™ BV symptoms RELIEF & TREATMENT gel used for application is very small, about the size of a penny. It adheres to the walls of the vagina and stays there for up to three days.

Can I use Replens™ BV symptoms RELIEF & TREATMENT gel during my period?

Yes, but its not recommended for best results.

How often should I use Replens™ BV symptoms RELIEF & TREATMENT gel?

As it is effective for up to 3 days, it is not necessary to use it more frequently.

Use when you feel BV symptoms for relief and treatment.

For prevention, you can also use at first signs or when you know by experience that it may reoccur (e.g. after period, after unprotected sex, before or after sex with new partner, etc.).

What are the ingredients contained in Replens™ BV symptoms RELIEF & TREATMENT gel?

INGREDIENTS: Purified Water, Glycerin, Polycarbophil, Carbomer Homopolymer Type B, Benzyl Alcohol, Ethylhexylglycerin, Sodium Hydroxide, Tocopherol.

7. Discover other Replens™ products

Also available without prescription:

- Replens™ Long Lasting Vaginal Moisturiser (x6 single-use pre-filled applicators)

Made in Canada.

Replens and Replens MD are trademarks of Church & Dwight Co., Inc.

Church & Dwight Co., Inc.
500 Charles Ewing Blvd,
Ewing Township, NJ 08628, USA

For further enquiries, please contact:
Church & Dwight UK Ltd.,
Wear Bay Road, Folkestone, Kent,
CT19 6PG, United Kingdom

SOFIBEL-Laboratoires Fumouze,
110-114 rue Victor Hugo,
92686 Levallois-Perret Cedex-France.

Do not re-use

Manufacturer

Authorised representative in the European Union

Importer

Distributor

Store between 15°C and 30°C

Consult instructions for use

Caution

Medical Device

Do not use if package is damaged and consult instructions for use

108333
REIN-53580-03UK
Rev 08-2023